# GSH vs GXM

Dr Fatmawati Kamal

CES

15.3.2024

# Introd

- Blood re details.
- Request
- Samples
- GSH – g
- GXM

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

..

# GSH


- Blood grouping – ABO Rh
  - Room temperature.
  - Forward & reverse
  - Column agglutination (Gel card), tube. Others: tile (forward) & microtiter plate.
- AB screening – refer PDN 8.3
  - Tube method – room T, 37°C, AHG.
  - Gelcard – 37°C, AHG incorp.

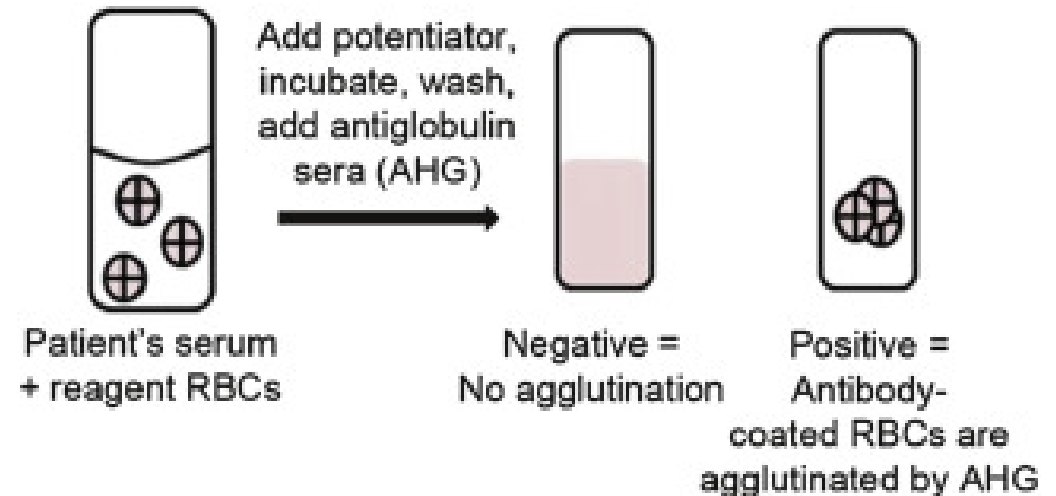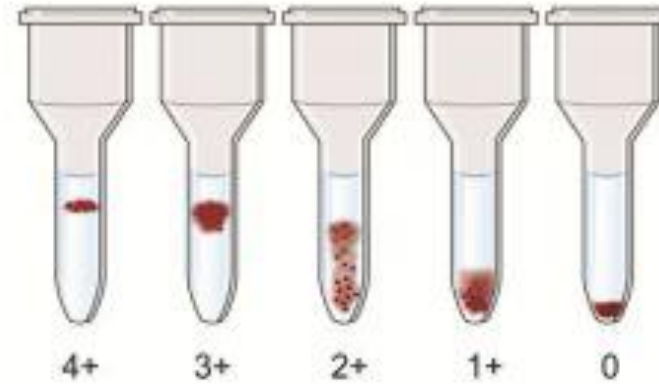# GSH – Screening cell panel

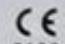**IVD**  0123

**Antikörper-Suchtest / Antibody screening / Recherche d'anticorps / Screening anticorpale / Escrutinio de anticuerpos irregulares / Teste pesquisa de anticorpos**

Antigen-Tabelle / Antigen-Table / Table d'antigènes / Tabela antigenica / Tabla de antígenos / Tabela de antígenios

**ID-DiaCell I-II-III Asia**


Rh-hr		Möglicher Genotyp Probable Genotype Genotype probable Probabile genotipo Genótipo provável	Spender Donor Donneur Donatore Donante Dador	Rh-hr						Kell				Duffy		Kidd		Lewis		P	MNS				Luth.		Xg		Spez. Antigene Special types Antigènes part. Antígenos parciales Otros Antígenos Tipos especiales	Resultat/Result/Résultat/ Risultato/Resultado/Resultado			
				D	C	E	c	e	C <sup>W</sup>	K	k	Kp <sup>a</sup>	Kp <sup>b</sup>	Js <sup>a</sup>	Js <sup>b</sup>	Fy <sup>a</sup>	Fy <sup>b</sup>	JK <sup>a</sup>	JK <sup>b</sup>	Le <sup>a</sup>	Le <sup>b</sup>	P <sub>1</sub>	M	N	S	s	Lu <sup>a</sup>	Lu <sup>b</sup>					Xg <sup>a</sup>
I	CCD.ee	R <sub>1</sub> R <sub>1</sub>	070460	+	+	0	0	+	0	+	+	0	+	nt	nt	+	0	+	0	+	0	0	+	+	+	+	0	+	+	N/A			
II	ccD.EE	R <sub>2</sub> R <sub>2</sub>	549821	+	0	+	+	0	0	0	+	0	+	nt	nt	0	+	0	+	+	0	0	+	+	0	+	0	+	+	N/A			
III	ccD.EE	R <sub>2</sub> R <sub>2</sub>	411996	+	0	+	+	0	0	0	+	0	+	nt	nt	+	0	0	+	0	+	+	0	+	0	+	0	+	+	N/A			

Eigenkontrolle, Autokontrolle / Autocontrol / Autocontrol / Autocontrol


**Mi(a+)**

## DAT & IAT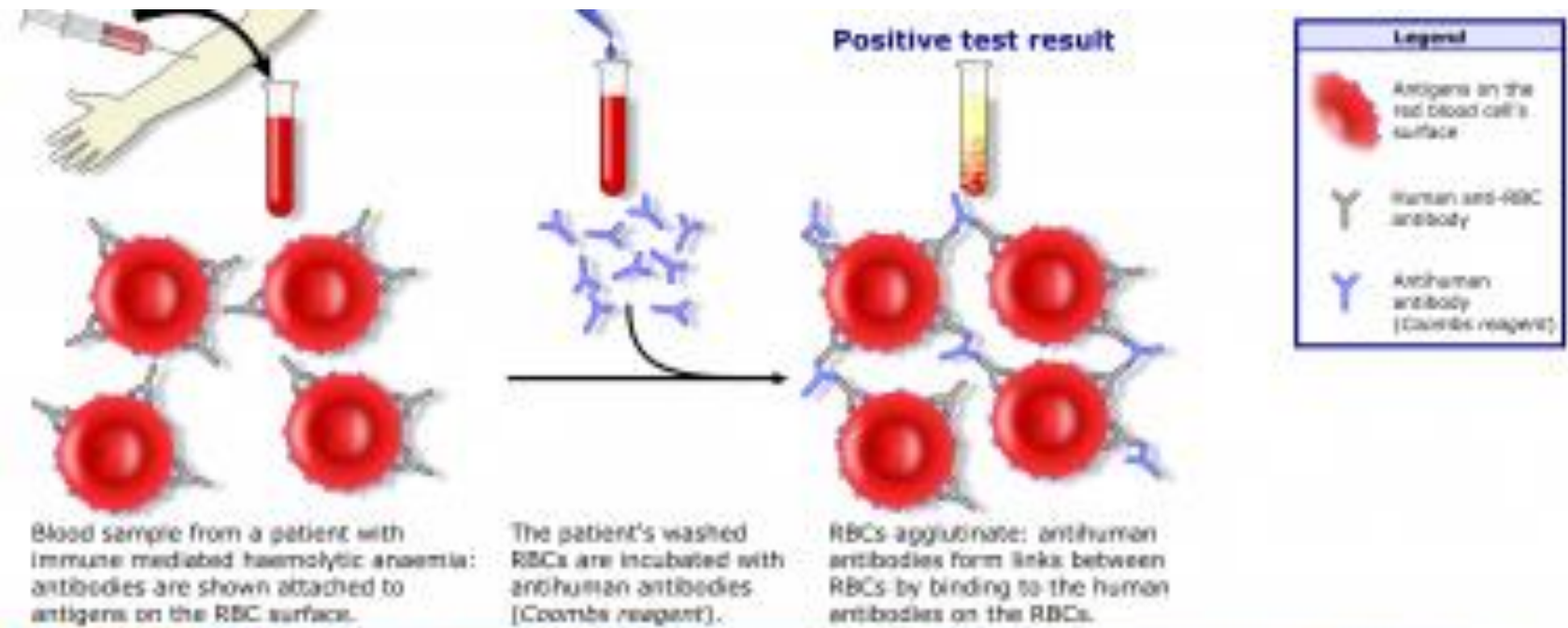


### Indirect Coombs test / Indirect antiglobulin test
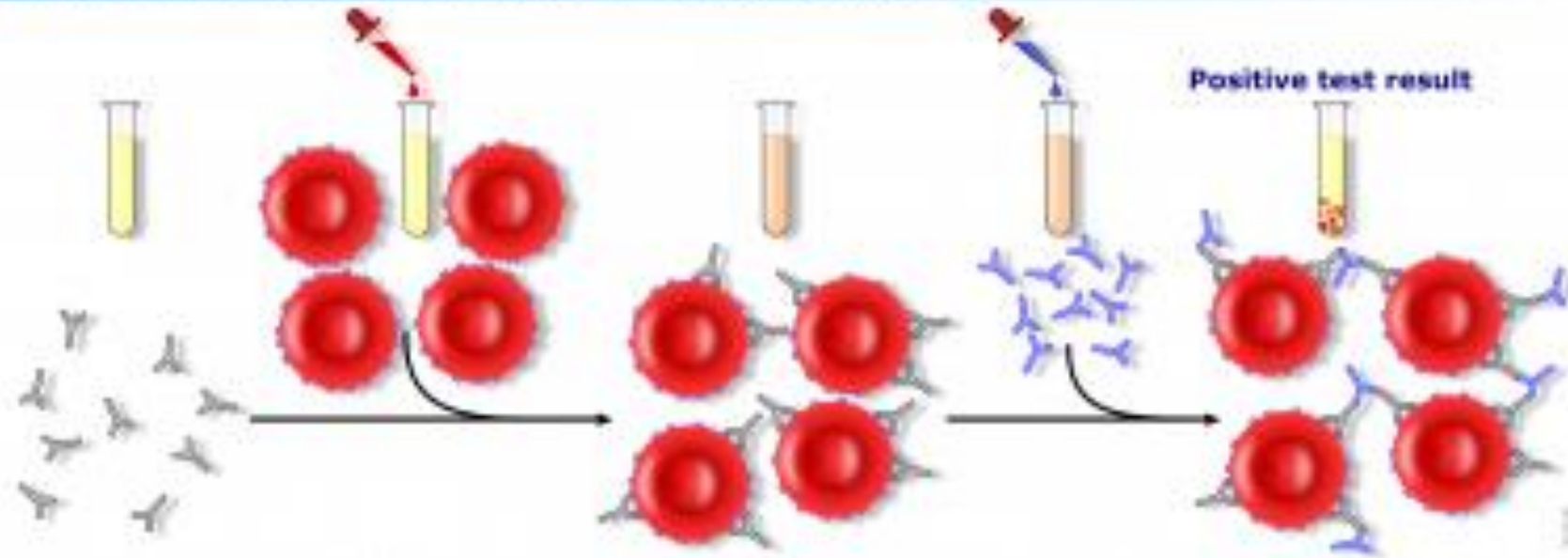
# Monospecific DCT
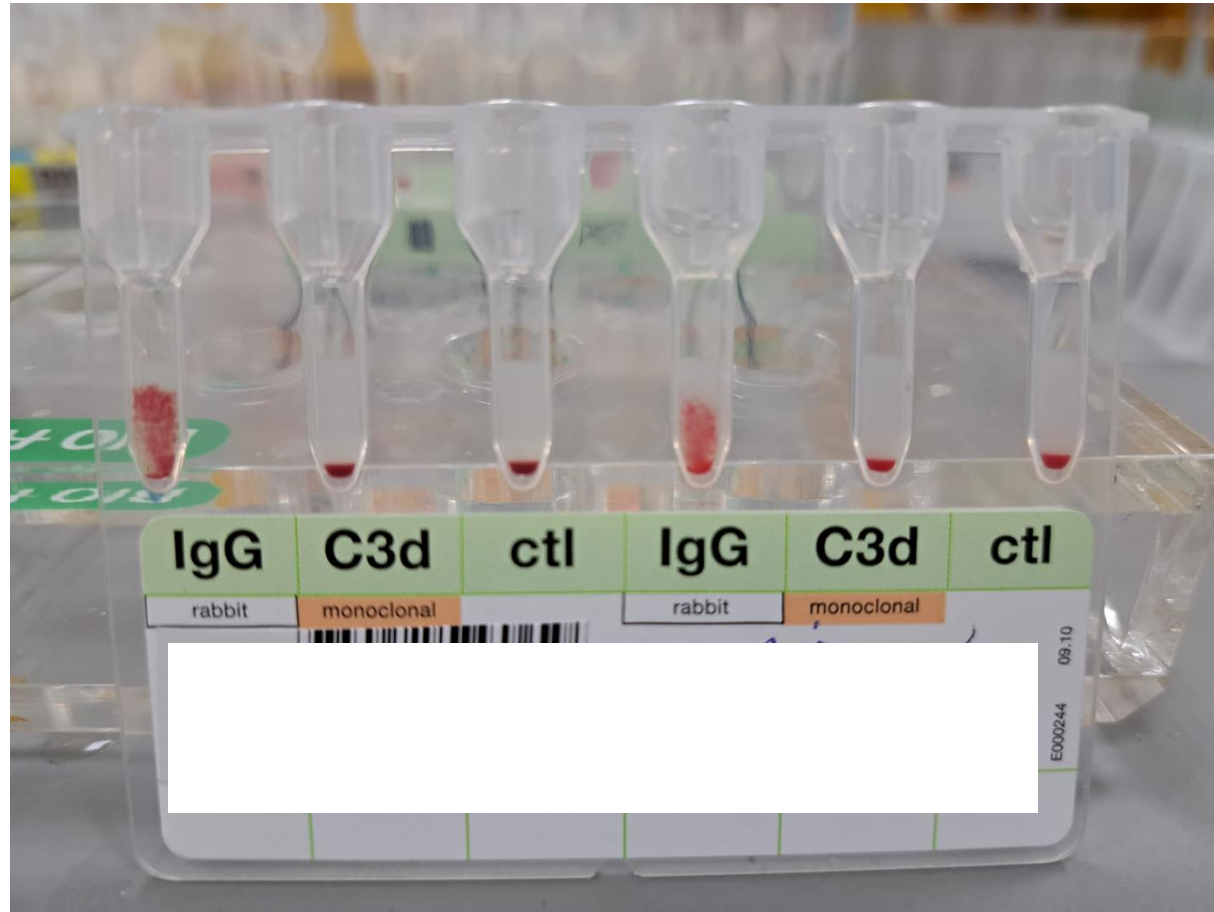
# AB Identification

- 10ml of blood in EDTA tube and 10ml blood in plain tube accompanied by a duly completed request form.
- History: prev. transfusion, pregnancy

[illegible]

# How to convert GSH to GXM

- CTC

- GSH to GXM conversion
- fill up the 'GSH to GXM Conversion Slip for PPUITM Sg Buloh' form & send to blood bank.
- Inform pathologist to convert in urgent cases.

- HASA

- 'paperless'
- If conversion require because of procedure/ op, kindly state the date of the procedure/ op.
- May allow convert within 3 days of GSH order.
- In urgent, please call CDL MO on-call to convert GSH -> GXM.
- Request in UNIMEDS -> transfusion med tab -> choose blood products
- Inform blood bank

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
(Huruf besar)

# GXM

- Tube method
  - Room temperature,
  - 37° C, and
  - AHG.
- Est. 2 hours or 1 hour
- Gel card

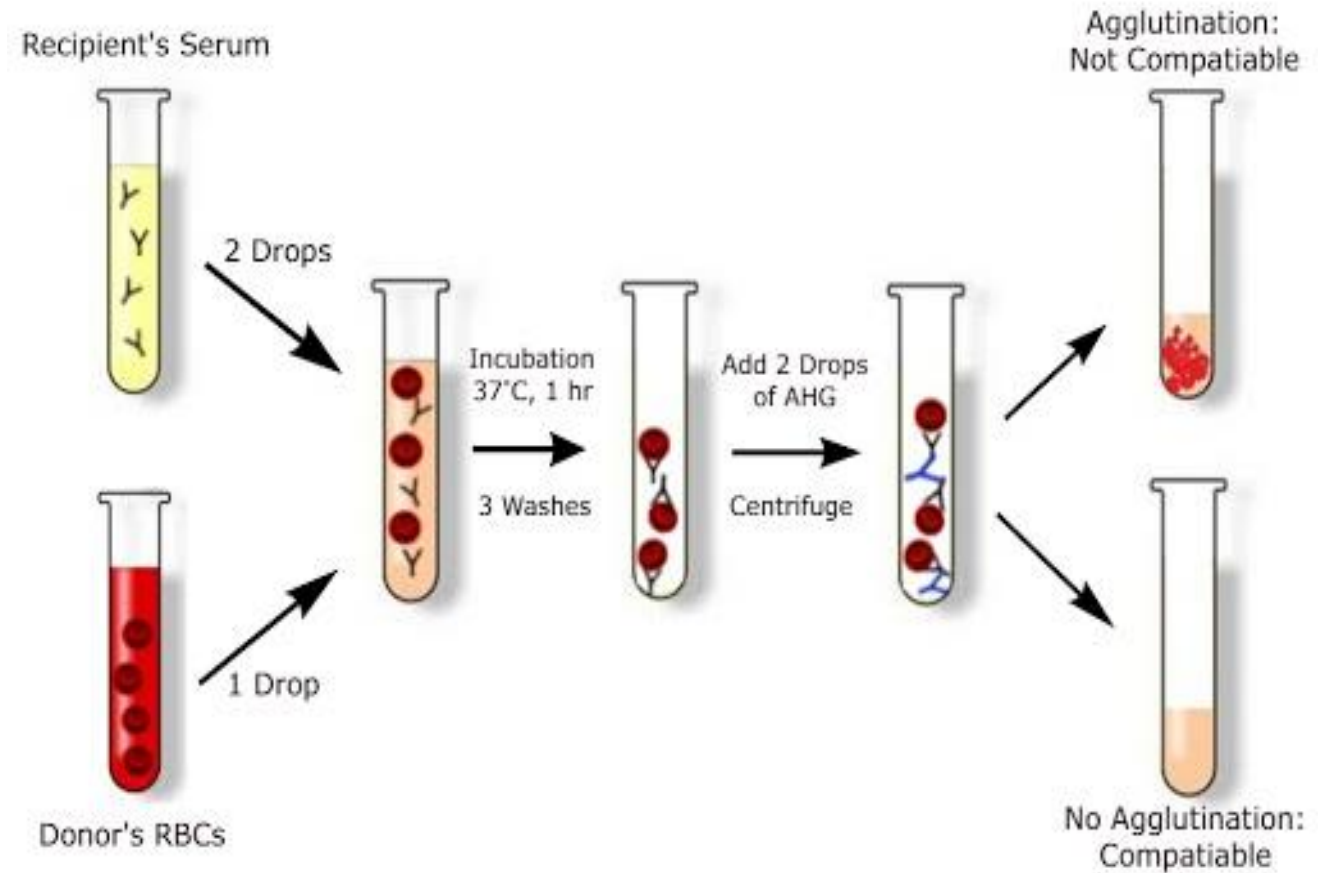# Questions 1

What constitute a GSH?

- i. Blood grouping
  - ii. Antibody identification
  - iii. Direct comb's test
  - iv. Antibody screening
- A. i & iv
  - B. i & iii
  - C. ii & iv
  - D. All of the above.

## Question 2


Choose a situation when a fresh blood sample is **NOT** required?

- A. Ordering FFP for the second time within the same admission.
- B. Ordering packed cell for the second time within the same admission.
- C. Ordering platelet only for the first time in current admission.
- D. Ordering GSH during antenatal clinic.


# Sample Rejection HTMU

JULY – DEC 2023

## Sample Rejection rate %

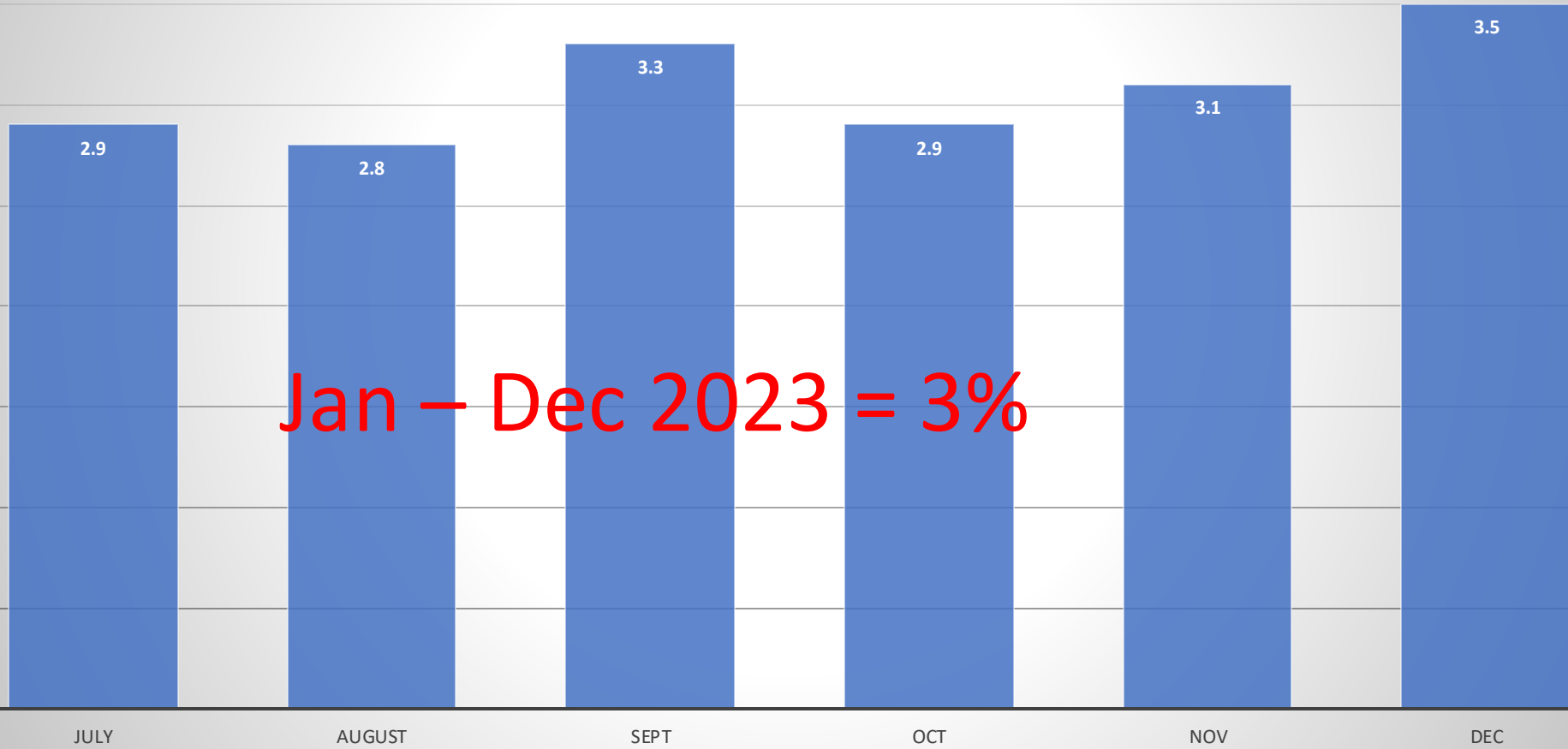
## Cause for sample rejection Jul - Dec 2023
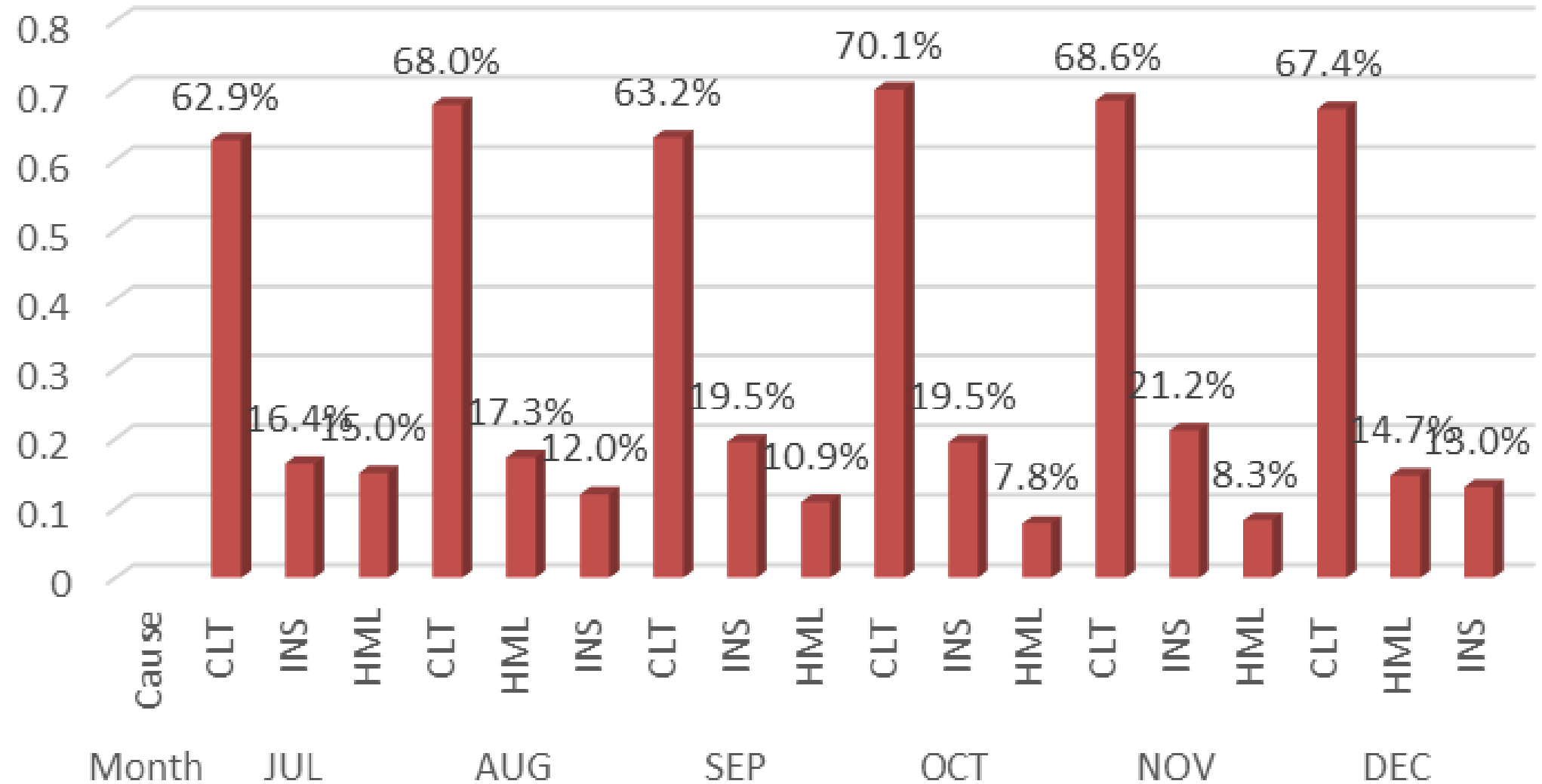
## Causes of Sample rejection Jul-Dec 2023

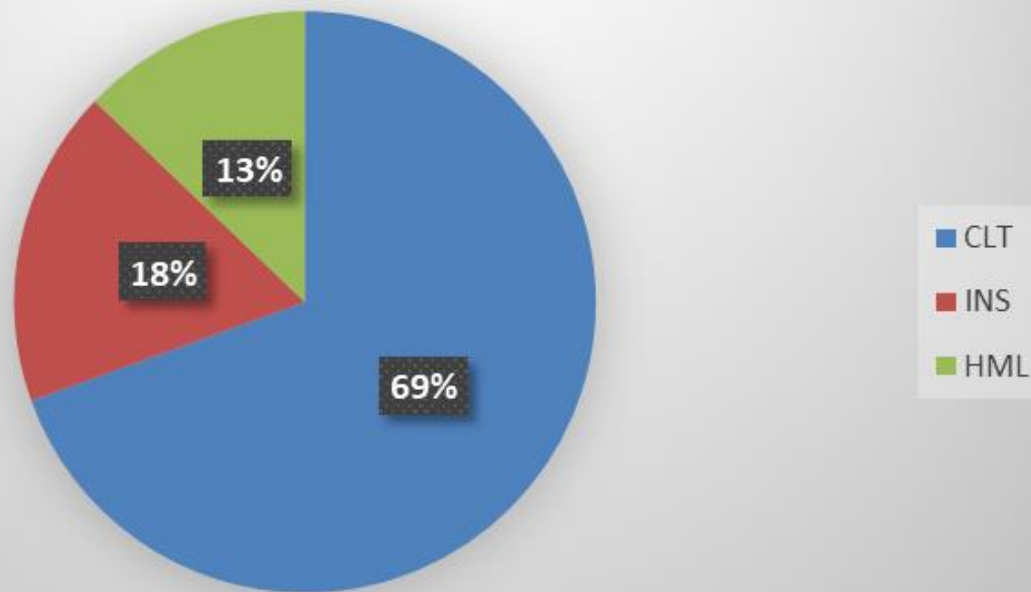JUL	1. ED
	2. PHLEBOTOMY WARD 8C
	3. PHLEBOTOMY WARD 8B
AUG	1. ED
	2. PHLEBOTOMY WARD 8B
	3. PHLEBOTOMY WARD 9A
SEP	1. ED
	2. PHLEBOTOMY WARD 8A
	3. PHLEBOTOMY WARD 9A
OCT	1. ED
	2. PHLEBOTOMY DAYCARE
	3. PHLEBOTOMY WARD 7A
NOV	1. ED
	2. PHLEBOTOMY DAYCARE
	3. PHLEBOTOMY WARD 9A
DEC	1. ED
	2. PHLEBOTOMY ICU
	3. PHLEBOTOMY WARD 9A

THANK YOU